

LIFELINKcentral[™] AED Program Manager

Contents

- Instructions to log an inspection for one AED at a time.
- Instructions to log an inspection for all AEDs at once.

To log an inspection for one AED at a time:

1 Visually confirm the AED(s) are ready for use based on the associated manufacturer operating instructions

2 Log in

Log into LIFELINKcentral AED Program Manager.

3 Select site

From "Sites," select the site for which you would like to log the inspection (Redmond-North Building).


The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the logo, user name 'Riggs, Malorie', and organization 'Sample Organization TL / All Sites'. A left sidebar shows 'Alerts' and 'My Account' sections. The main content area is titled 'Sites' and features a search bar. Below the search bar, two site entries are listed:

- Redmond-North Building (\$S774426)**: Address: North Building, Redmond, Washington, 98052; Primary Site Coordinator: Riggs, Malorie; Phone Number: 425-867-4700; E-mail: tammy.lombardi@stryker.com; Status: NEEDS ATTENTION (DEVICE)
- Redmond-South Building (\$S776691)**: Address: South Building, Redmond, Washington, 98052; Primary Site Coordinator: Riggs, Malorie; Phone Number: 425-867-4700; E-mail: tammy.lombardi@stryker.com; Status: READY

A purple arrow points from a 'Select Site' button to the 'Redmond-North Building' entry.


4 Click the checkmark to record an inspection

[Back to contents](#)

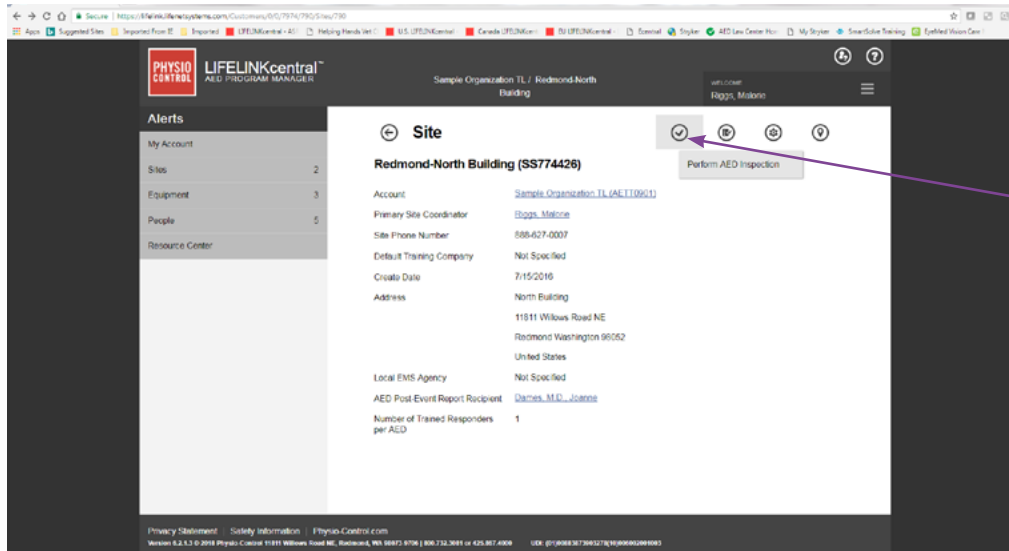

5 Select AED

Select the AED for which you are recording the inspection:

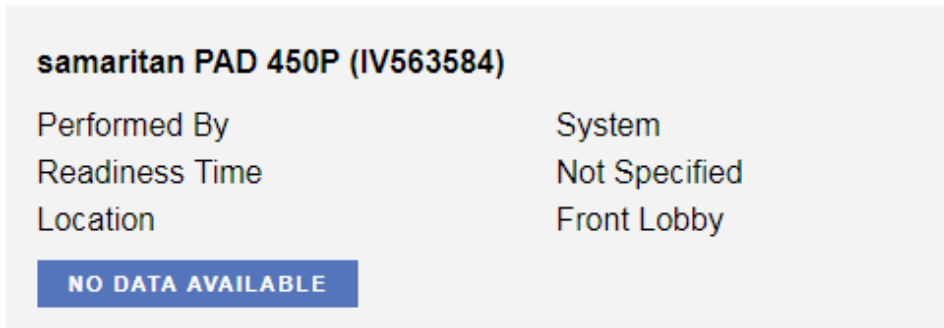

6 Verify information (Important for first inspection, optional for future inspections)

Expand each section and verify the information.

Perform Inspection ✕

samaritan PAD 450P (IV563584)

- EQUIPMENT INFORMATION ∨
- ACCESSORIES ∨
- INSPECTION CHECKLIST ∧

Device Present	<input checked="" type="checkbox"/>
Visible Damage	<input type="checkbox"/>
Checklist state	Ready ∨
Readiness Status	Ready

- NOTES ∨

7 Confirm AED readout

[Back to contents](#)

Proceed to Inspection Checklist and confirm AED is ready for use:

Perform Inspection ✕

samaritan PAD 450P (IV563584)

- EQUIPMENT INFORMATION ∨
- ACCESSORIES ∨
- INSPECTION CHECKLIST ∧

Device Present


Visible Damage

Checklist state Ready ∨

Readiness Status Ready

NOTES ∨

Submit Cancel



- Click the “Submit” button to submit your entries.
- The AED status will turn green for “Ready” confirming a successfully logged inspection:

samaritan PAD 450P (IV563584)

Performed By Riggs, Malorie

Readiness Time 3/15/2018 3:01 PM

Location Front Lobby

READY

You will be returned to the list view of the AED Inspection Schedule page. From there, repeat the steps above to perform a maintenance inspection on the next AED in the list.

To log an inspection for all AEDs at once:

8 Visually confirm the AED(s) are ready for use based on the associated manufacturer operating instructions

9 Log in

Log into LIFELINKcentral AED Program Manager.

10 Inspections

From "Equipment," click "Inspections" to see the inspection schedules for those AEDs for which you are responsible.

The screenshot displays the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the logo, user information (Sample Organization TL / All Sites, WELCOME Riggs, Malorie), and a menu icon. The left sidebar contains navigation options: Alerts, My Account, Sites (2), Equipment (6), Manage Equipment, a donut chart showing 'AEDs Ready 4 of 6', a status legend (Ready: 4, No Data Available: 2), People (8), and Resource Center. The main content area is titled 'Equipment' and features a search bar. It lists equipment for two buildings: 'Redmond-North Building (SS774426)' and 'Redmond-South Building (SS776591)'. Under the North Building, a 'Lifeline™ AED (GYdi685)' is shown with a dropdown menu open, highlighting 'Inspections'. Other items listed include 'Device Issues', 'Custom Equipment', 'Accessories', and 'AED Usage Log'. The South Building section shows 'samaritan PAD 450P (IV563584)' and 'LIFEPAK CR Plus (PC34344088)'. Two purple callout boxes with arrows point to the 'Equipment' header and the 'Inspections' menu item.

11 Inspection Schedules

Next, click on the Inspection Schedule to which you would like to navigate. In this example, we will click on “Redmond-North Building” to log an inspection for all AEDs affiliated with that schedule:

Inspection Schedules

Redmond-North Building (SS774426)

Navigate to schedule

AED Inspection Schedule

Person Responsible	Riggs, Malorie
Frequency	Monthly
Last Inspection Date	3/15/2018

Redmond-South Building (SS776691)

AED Inspection Schedule

Person Responsible	Johnson, Robert
Frequency	Monthly
Last Inspection Date	3/9/2018

12 AED Inspection Schedule

Once you are on the landing page for the “Redmond-North Building” Inspection Schedule, click the “Tool” icon to the right of the words “AED Inspection Schedule,” and then click the “Check” icon to log a check for all AEDs at once.

The screenshot shows the LIFELINKcentral AED Program Manager interface. The main content area is titled "AED Inspection Schedule" and lists three AEDs:

- Defibtech Lifeline™ AED (GYdi685)**
Performed By: System
Readiness Time: Not Specified
Location: Shipping Dock
Status: NO DATA AVAILABLE
- Philips HeartStart FRx (SXsd4568)**
Performed By: System
Readiness Time: Not Specified
Location: Break Room
Status: NO DATA AVAILABLE
- samaritan PAD 450P (IV563584)**
Performed By: Riggs, Malorie
Readiness Time: 3/15/2018 3:01 PM
Location: Front Lobby
Status: READY

A toolbar at the top right of the AED list contains several icons. A purple callout box labeled "Tool icon" points to the top-right icon, and another purple callout box labeled "Check icon" points to the bottom-left icon (a checkmark).

13 Confirm

The following prompt will appear, asking you to confirm you would like to update the readiness status of all AEDs associated with this schedule to “Ready.” Click “Yes” to complete the check:

Complete Inspection

The readiness status of all Equipment not previously recorded today will be set to 'Ready' with today's date.
Do you want to proceed?

A confirmation dialog box titled "Complete Inspection" with a close button (X) in the top right. The dialog asks "Do you want to proceed?" and has two buttons: "Yes" and "No". A purple callout box labeled "Click Yes" points to the "Yes" button.

14 Completed

[Back to contents](#)

You will be routed to the list view for all AEDs on that Inspection Schedule. The maintenance inspection for all AEDs listed in this schedule is complete, and all AEDs will show green for “Ready” with today’s date; in this example, 3/15/2018:

AED Inspection Schedule

Enter text to search the list below

samaritan PAD 450P (IV563584)

Performed By Riggs, Malorie
Readiness Time 3/15/2018 3:01 PM
Location Front Lobby

READY

Defibtech Lifeline™ AED (GYdi685)

Performed By Riggs, Malorie
Readiness Time 3/15/2018 3:15 PM
Location Shipping Dock

READY

Philips HeartStart FRx (SXsd4568)

Performed By Riggs, Malorie
Readiness Time 3/15/2018 3:15 PM
Location Break Room

READY

Physio-Control is now part of Stryker.

For further information, please contact your Account Support Representative or call 888-627-0007